



Oasis Series

# Y6y8c

## ELECTRIC BED

Registration code: SKD-C




**SAIKANG** / Medical

Healthier  
your life

# Y6y8c

## FEATURES AND HIGHLIGHTS

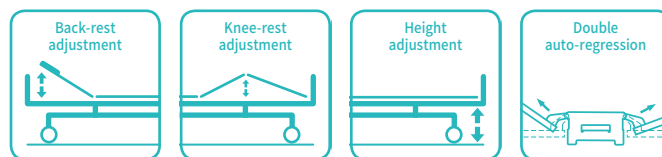

### Technical parameters

▪ External size (LxW)	2130x1050 mm
▪ Mattress platform	1925x880 mm
▪ Safe working load	250 kg
▪ Height adjustment	360-800 mm
▪ Back-rest adjustment	0-75° (±10°)
▪ Knee-rest adjustment	0-35° (±10°)
▪ Auto-regression:	100+60 mm
▪ Angle indicator	0-90°
▪ Position line	30°
▪ Voltage and frequency	100-240 V · 50/60 Hz
▪ Plug as required (hospital grade)	

### Optional


<input type="checkbox"/> Weighing system
<input type="checkbox"/> Bed extension 20cm
<input type="checkbox"/> Back-up battery
<input type="checkbox"/> Nurse control panel
<input type="checkbox"/> Emergency stop button
<input type="checkbox"/> Cardiac chair position
<input type="checkbox"/> Directional wheels function
<input type="checkbox"/> 5th and 6th extra side rail for full protection
<input type="checkbox"/> Monkey pole
<input type="checkbox"/> Balkan frame
<input type="checkbox"/> Linen holder
<input type="checkbox"/> One-button for patient mobilisation position
<input type="checkbox"/> One button nursing position
<input type="checkbox"/> Lowest position indicator

### Technical configuration

<input checked="" type="checkbox"/> Linear motor	3 pcs
<input checked="" type="checkbox"/> ABS Oasis side rail extra height	1 set
<input checked="" type="checkbox"/> ABS Oasis bed ends with safe lock	1 set
<input checked="" type="checkbox"/> PP bed platform, removable & washable	9 pcs
<input checked="" type="checkbox"/> Plastic socket inside pp guardrail for phone or glasses	2 pcs
<input checked="" type="checkbox"/> Patent angle indicator for back-rest and trendelenburg position	4 pcs
<input checked="" type="checkbox"/> Ø70mm wall bumper wheel	4 pcs
<input checked="" type="checkbox"/> IV pole	1 set
<input checked="" type="checkbox"/> IV pole prevision	4 pcs
<input checked="" type="checkbox"/> Drainage hook	2 pcs
<input checked="" type="checkbox"/> Central locking pedal	2 pcs
<input checked="" type="checkbox"/> ABS mattress stopper	2 pcs
<input checked="" type="checkbox"/> Hand controller	1 pc


#### Great multi-care

Respects easy patient mobilisation:  
One-button for patient mobilisation position -back plate raised, double hole design on back part guardrail, which supports patient self standing.

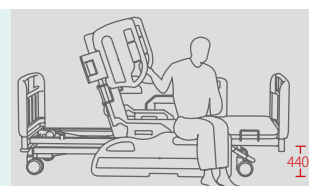

#### Double regression function

Auto-Regression 10+6 cm  
When repositioning the backrest and thigh rest, the Y8y makes the back plate retreat 10cm and the thigh plate retreats 6cm so to minimises the pressure and shear forces in the patient's pelvic region. This results in a significant reduction in the development of pressure ulcers.

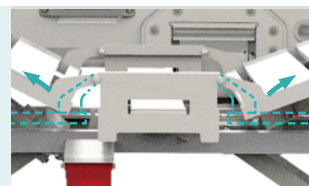


**Electric motor**

Linear motor, with UL/ROHS/EN standard. Noise under 50 dB, with lifetime 20,000 times movement.

**Joint-free technology**

Removable plastic covers formed from 9 removable blow moulded lamella to minimise the slots where unwanted infections may hide.

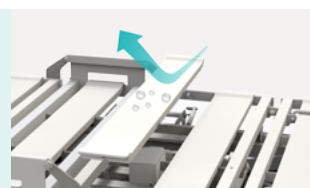**Wheels**

Castors with TPR tire, no worn-out after running 30KM, united forming without bolts.  
Pass Dynamic test: Bearing 120kg run 30KM and pass obstacles 500time.

**Brake**

Three position braking system, transporting can maintain a straight line.

**Tank**

Socket on side rail for placing the patient cellphone or glasses.

**Angle indicator**

Patent angle indicator, clearly show back-rest and tredelenbug position.

**Head & foot board**

Patented design Detachable head & foot board, using the eccentric locking structure, one step quick removable.

**Safety guardrail**

The designed guardrail fully considered to provide enough space and hand grasp for patient to get out of bed, and provide extremely high safety. Two steps unlocking system, to prevent patients from accidentally lowering the guardrail (patented design).

**Handheld controller**

Handheld controller for patients and nurses, ergonomic design. Built-in LED torch, backlit keyboard design for night use.

**Wall bumper wheel**

Φ70mm bumper wheels at the outermost periphery of the bed to protect the bed and wall, anti-collision and durable.

**IV pole**

Stainless steel column with plastic hook.

**Drainage hooks**

Mobile plastic drainage hooks.







## CONSTANT INNOVATION



Design



Research



Manufacturing



Maintenance



Service



## WORLDWIDE BRANCHES

### USA

**Saikang Medical Technology, Inc.**  
17800 Castleton St Ste 180,  
City of Industry, 91748, California.  
Tel.: +1 949648 2776

### United Kingdom

**UK Healthward International Ltd.**  
Dept. 111, 196 High Road Wood  
Green, N22 8HH, London.  
Tel.: +44 2032399738

### Russia

**Saikang Medical Russia LLC**  
Truda 174,  
454080 Chelyabinsk.  
Tel.: +7 (351) 214 5566 / 8811 / 1516

### East Africa

**Saikang Medical Solutions Ltd.**  
Viraj Complex Unit 3, Mombasa  
Road, P.O. Box 14262-00800, Nairobi.  
Kenya.  
Tel.: +254 733 704 690

### Middle East

**Saikang Medical Supplies LLC**  
X22-S03 England Cluster,  
International City, Dubai.  
United Arab Emirates.  
Tel.: +20 128 8962497

**SAIKANG** Medical

Jiangsu Saikang Medical Equipment Co., Ltd.  
STOCK CODE: 870098

📍 No. 35, Lehong Road · Zhangjiagang (Jiangsu) · China  
☎ +86 18021231901 · 🕒 24/7  
✉ sales@chinasaikang.com  
🌐 www.saikangmedical.com



Company video



General catalogue